**STERİL VE TEK KULLANIMLIK HİPODERMİK ŞIRINGA VE STERİL İĞNE UCU
KULLANIM KILAVUZU /
STERILE AND SINGLE USE HYPODERMIC SYRINGE AND STERILE NEEDLE TIP
USER MANUAL**

AÇIKLAMA / DESCRIPTION

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Genject Hypothermic Syringe products consist of the main components of plunger and body and have sealed/unsealed, needle/needleless options.

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Disposable hypodermic syringes manufactured by injection method from high molecular weight polypropylene and polyethylene raw material are assembled with needle and isoprene rubber manufactured from SUS304 stainless material in accordance with ISO 5832-1 & ASTM F138 standard and the final product is obtained.

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The Sterile Hypodermic Syringe for Single Use with / without Needle is intended to be used for medical purpose to inject fluid into or withdraw fluid from body.

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Hypodermic Syringe is indicated for surgical use, vaccination, medical diagnosis and treatment and surgical applications related to blood vessels.

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Single Use Syringe is indicated for surgical use, vaccination, medical diagnosis and treatment and surgical applications related to blood vessels.

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WARNINGS

- *Do not use products that are torn or whose package has been opened.*
- *Open the package from the "Open it here" part on the label.*
- *After removing the syringe from the package, check for cracks that may cause discharge.*
- *Remove the needle from the cover and draw the liquid, such as medicine, into the syringe.*
- *Make sure there is no air in the syringe.*
- *After use for injection, re-attach the needle cover to the needle and dispose of it in a safe disposal bin.*
- *Do not reuse.*
- *Do not sterilize.*
- *Observe the warnings on the label information before using the syringe.*

- *Use in accordance with the expiry date.*
- *Do not use the product that has expired.*
- *Do not use together with different devices other than the intended use and do not use outside the intended purpose.*
- *Make sure the cover is not broken or cracked when removing the needle from the cover.*
- *Make sure the needle tip is not burred or blunt.*

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When preparing the syringe for use, the syringe package is opened from the "open it here" part on the label.

Hypodermic syringes for single use;

To attach the needle, it is removed from the protective shield and gently pushed from the conical head of the syringe.

It is checked whether the needle is properly connected to the conical head.

The liquid to be injected is taken from the needle part into the body chamber by pulling the plunger back.

The fluid is injected by pushing the plunger forward again with the help of the body flanges.

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Storage and conditions of use shall be such as to guarantee that the device is in perfect condition and packaged. No damaged products should be used. The expiry date given on the label guaranteeing the sterility of the device must be checked. It is important to check the integrity of the package.

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This device is supplied sterile. Check the package label to learn more. The device has been sterilized by ethylene oxide sterilization (ETO) in accordance with current standards. The ETO label icon on the label and/or sterilization packaging shall be in green color.

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Devices must be stored in a dry place in their original unopened packaging in a range of 25-5°C, 10-85% RH and must not be used after the expiry date.

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It can be used for all age groups. It is designed for use by healthcare professionals and clinical support specialists.

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To request additional information, please refer to the contact information listed in this document.

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